



2019–Novel Coronavirus (2019-nCoV) IgG/IgM GICA Rapid Test

(Colloidal Gold)(Serum /Plasma/ Whole Blood)




Easier:Quick sampling by fingertip blood.

Rapid:Result in 10-15 min.

Accurate:Results with IgG and IgM respectively.

Application:For suspicious patients with symptoms, mild symptoms, or even without symptoms, also for testing people with close contact of infected patients and people under quarantine control.

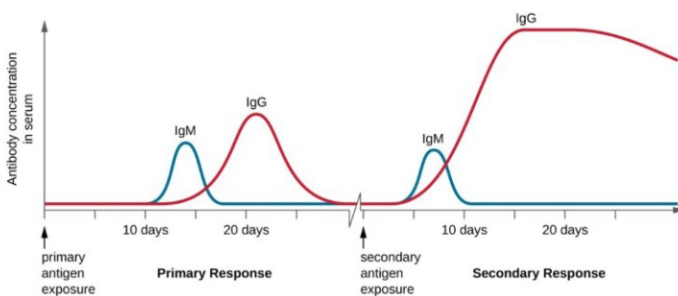

Bioeasy 2019-nCoV IgG/IgM GICA Rapid Test Kit, can quickly detect the presence of IgG and IgM, respectively, which can indicate the infection period of patients.

According to the Novel Coronavirus Pneumonia Diagnosis and Treatment Plan (7th Edition) published by General Office of National Health Committee, serology testing can be used as a diagnostic method of COVID-19 infection.

By differentiating the test results of IgM and IgG , which is very important, can help the doctors identify it's early infection, or middle/late infection, so that doctors can take different treatments for those patients.

Test Procedure

Test Procedure




Disinfect finger with an alcohol pad




Pierce the disinfected finger with a blood




Collect blood with a dropper

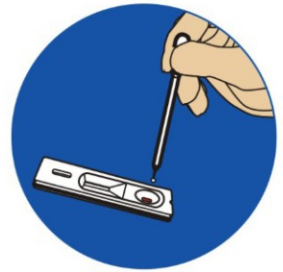


Add 1 drop of fingertip blood into the sample well

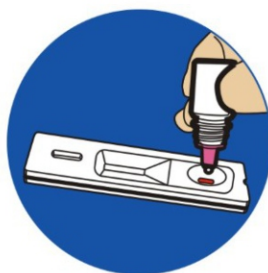

Add 2 drops of sample dilution into the sample well slowly

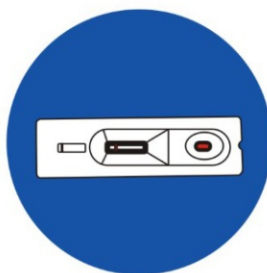

Read the result in 10-15 min

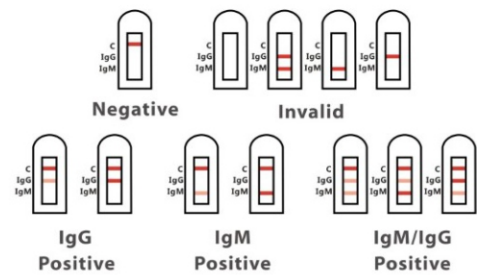

If there is visible red line of the test line, it indicates a positive result of the specific line

Specification

Cat. No.	Product	Format	Specimen	Pack	Qualification
YRLG22301025	2019-Novel Coronavirus (2019-nCoV) IgG/IgM GICA Rapid Test (Colloidal Gold)	Cassette	WB/S/P	25T	CE